
MC 300® NEBULIZER


Quick Start Guide

How to use your nebulizer.

1 Wash your hands.

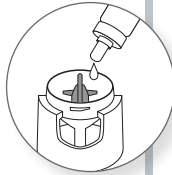2 Carefully add medicine to the cup.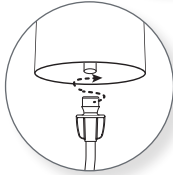


3 Attach the tubing and turn on your air source.

4 Seal lips around the mouthpiece (or ensure a good seal around the edges of the mask).

5 Breathe through the device until the mist stops.

Clean your nebulizer after each treatment and replace after 7 days use.

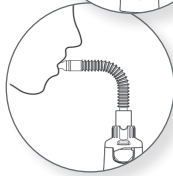These are abbreviated instructions for use, please remember to read the complete instructions that are packaged with your nebulizer.

What is a Nebulizer?

A nebulizer uses an air source to convert liquid medications into a fine mist, so you can inhale it into your lungs. A mouthpiece or a mask will be used, and you will simply breathe until the mist stops. It is important to take your treatments as prescribed, even if you feel fine. Talk to your healthcare team if you have questions about your medication.

Medication Name	Fill Amount	Treatment Frequency	1 x day 2x day	Purpose of Medication

If you have questions about the prescribed medicine inside the nebulizer, please talk to your healthcare team.

Words or phrases accompanied by ™ and ® are trademarks and registered trademarks of Monaghan Medical Corporation or an affiliate of Monaghan Medical Corporation. © 2018-2021 Monaghan Medical Corporation. † trademarks and registered trademarks of the respective companies.

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